



**Germo**®  
S.P.A.

Capitale Sociale € 156.000,00 interamente versato - R.E.A. 497966 Cod. Fisc. e Part. IVA 00772350153 - Iscr. 125750 R.I. Canc. Trib. MI

Sede Legale Stabilimento e Uffici: 20032 CORMANO (Milano) Via Giotto, 19/21  
Telefono +39-0266301938 r.a. - Telefax +39-0266301939  
E-mail: info@germodis.com [www.germo.it](http://www.germo.it)

## GERMOCID LINE - TECHNICAL DATASHEET

### GERMOCID 2%

GERMOCID 2% is a Medical Device CE 0546 class IIb in compliance with Directive 93/42/CEE.

GERMOCID 2% is a ready to use solution with a detergent and disinfectant action for invasive medical devices and surgical instruments. GERMOCID 2% has a high efficiency as detergent and disinfectant also in presence of organic material (blood, saliva, organic exudates) and it can be used also on dental instruments. The dirt removal does not require mechanical action.

It is active against Gram-positive and Gram-negative bacteria, fungi, mycobacteria and viruses (included HIV-HBV-HCV).



#### PROPERTIES

GERMOCID 2% is a specific product for the cleaning, high-level disinfection and decontamination before the sterilization of surgical instruments and invasive medical devices. For professional use, as it is.

#### INSTRUCTIONS FOR USE

Soak In the solution the instruments to treat according to the table. Rinse thoroughly with water before use. It can also be used in ultrasound equipment.

#### CONTACT TIMES:

|                                   |            |
|-----------------------------------|------------|
| Bacteria                          | 15 minutes |
| Fungi                             | 15 minutes |
| Mycobacteria                      | 60 minutes |
| Viruses (Adenovirus, HIV-HCV-HBV) | 60 minutes |

#### PACKAGING

|                |                    |                           |
|----------------|--------------------|---------------------------|
| Internal Code: | Primary packaging: | Secondary packaging:      |
| F111           | 1000 ml bottle     | Box containing 12 bottles |

|               |                |                       |                        |
|---------------|----------------|-----------------------|------------------------|
| Barcode       | ITF            | Box size (mm)         | Pallet composition     |
| 8009110025509 | 08009110025509 | 335 w x 253 d x 274 h | 40 ctn (4 floors x 10) |

All primary packaging is made of high polyethylene density (HDPE) according to the specifications provided by the Pharmacopoeia. This material does not contain allergenic and is perfectly compatible with all components of the formulation.



**Germodis**®  
S.P.A.

Sede Legale Stabilimento e Uffici: 20032 CORMANO (Milano) Via Giotto, 19/21  
Telefono +39-0266301938 r.a. - Telefax +39-0266301939  
E-mail: info@germodis.com www.germodis.it

Capitale Sociale € 156.000,00 interamente versato - R.E.A. 497966 Cod. Fisc. e Part. IVA 00772350153 - Iscr. 125750 R.I. Canc. Trib. MI

## GERMOCID LINE - TECHNICAL DATASHEET

### GERMOCID 2%

#### COMPOSITION

100,0 g of product contain:

Benzalkonium chloride 2,0 g; O-phenylphenol 0,2 g; coformulants; depurated water q.s. to 100,0 g.

#### PRODUCT FORM

Colourless/light green liquid. No perfumed.

#### ACTIVITY

GERMOCID 2% is a ready to use solution with elevated effectiveness and rapidity as detergent and disinfectant. It is active against Gram-positive and Gram-negative, fungi, mycobacteria and viruses.

CONTACT TIMES:

| Norm                | Activity                          | Contact times |
|---------------------|-----------------------------------|---------------|
| EN13727-<br>EN14561 | Bacteria                          | 15 minutes    |
| EN13624-<br>EN14562 | Fungi                             | 15 minutes    |
| EN14348-<br>EN14563 | Mycobacteria                      | 60 minutes    |
| EN14476             | Viruses (Adenovirus, HIV-HCV-HBV) | 60 minutes    |

#### VALIDITY

3 years. The indicated period of validity refers to the product stored in its container and properly used and stored.

#### HOW TO STORAGE

Store in a cool, dry place, away from heat sources.

#### QUALITY CONTROL

The components (raw materials, containers, labels, etc.) and the processing steps of each lot are regularly and carefully checked internally following the procedures of the Organization Quality Management System certified UNI EN ISO 9001 and UNI EN ISO 13485.

#### AUTHORIZATIONS

Medical Device CE 0546 class IIb in compliance with Directive 93/42/EEC as amended by Directive 2007/47/EEC.

#### INFORMATION FOR PROFESSIONAL USER