



**Germo**®  
S.P.A.

Capitale Sociale € 156.000,00 interamente versato - R.E.A. 497966 Cod. Fisc. e Part. IVA 00772350153 - Iscr. 125750 R.I. Canc. Trib. MI

Sede Legale Stabilimento e Uffici: 20032 CORMANO (Milano) Via Giotto, 19/21  
Telefono +39-0266301938 r.a. - Telefax +39-0266301939  
E-mail: info@germodis.com [www.germo.it](http://www.germo.it)

## LINEA GERMOCID - TECHNICAL DATASHEET

### GERMOCID VERDE


GERMOCID VERDE is a Medical Device class IIb in compliance with Directive 93/42/EEC.

GERMOCID VERDE, ready to use, is a fast and practical disinfectant. The product has a high alcohol content, and it is perfect for disinfecting and aseptic conservation of invasive medical devices. The synergistic combination of its active ingredients guarantees maximum protection against fungi, viruses and Gram-positive and/or Gram-negative bacteria.

GERMOCID VERDE also acts in the presence of organic material.

Its formula has a high cleaning power and evaporates quickly, leaving however a pleasant fragrance.

Ideal for all surgical instruments because it preserves their sharpness.

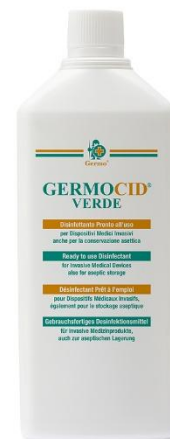

#### PROPERTIES

GERMOCID VERDE is used for disinfection and aseptic preservation of invasive surgical instruments. For professional use.

#### INSTRUCTIONS FOR USE

For disinfection and aseptic conservation of invasive medical devices, immerse them directly in a container containing the ready-to-use solution of GERMOCID VERDE. Leave for 60 minutes for a bactericidal, fungicidal and virucidal activity, keep immersed for aseptic conservation. Proceed with a thorough rinsing before use, the last of which with sterile water.

#### PACKAGING

Internal Code:	Primary packaging:	Secondary packaging:
T165	Bottle of 1000 ml	Box containing 12 bottles

Barcode	ITF	Box size (mm)	Pallet composition
8009110025073	08009110025073	343 w x 261 d x 259 h	40 ctn (4 floors x 10)

All primary packaging is made of high polyethylene density (HDPE) according to the specifications provided by the Pharmacopoeia. This material does not contain latex and is perfectly compatible with all components of the formulation.



**Germo**®  
S.P.A.

Capitale Sociale € 156.000,00 interamente versato - R.E.A. 497966 Cod. Fisc. e Part. IVA 00772350153 - Iscr. 125750 R.I. Canc. Trib. MI

Sede Legale Stabilimento e Uffici: 20032 CORMANO (Milano) Via Giotto, 19/21  
Telefono +39-0266301938 r.a. - Telefax +39-0266301939  
E-mail: [info@germodis.com](mailto:info@germodis.com) [www.germo.it](http://www.germo.it)

## LINEA GERMOCID - TECHNICAL DATASHEET

### GERMOCID VERDE

#### COMPOSITION

100,0 g of product contain: Chlorexidine Digluconate 0,1 g; Benzalkonium Chloride 0,1 g; Ethilic alcohol 45,0 g; Iso-propyl alcohol 15,0 g; coformulants, depurated water q.s. to 100,0 g.

#### PRODUCT FORM

Green Liquid with characteristic perfume.

#### ACTIVITY

The combination of Quaternary Ammonium Salts and Chlorexidine and the elevated alcohol content, guarantees a wide range of rapid and prolonged action against Gram-positive and Gram-negative bacteria, fungi and viruses.

#### TEST

EN13727; EN13624; EN14561; EN14562; EN14476.

#### WARNINGS

Replace the solution to be used for the treatment of instruments each week.

#### VALIDITY

3 years. The indicated period of validity refers to the product stored in its container and properly used and stored.

#### HOW TO STORAGE

Store in a cool, dry place, away from heat sources.

#### QUALITY CONTROL

The components (raw materials, containers, labels, etc.) and the processing steps of each lot are regularly and carefully checked internally following the procedures of the Organization Quality Management System certified UNI EN ISO 9001 and UNI EN ISO 13485.

#### AUTHORIZATIONS

Medical Device CE 0546 class IIb in compliance with Directive 93/42/CEE as amended by Directive 2007/47/EEC.

**CONFIDENTIAL INFORMATION FOR HEALTH WORKERS AND PROFESSIONAL USERS**