



Germodis®
S.P.A.

Capitale Sociale € 156.000,00 interamente versato - R.E.A. 497966 Cod. Fisc. e Part. IVA 00772350153 - Iscr. 125750 R.I. Canc. Trib. MI

Sede Legale Stabilimento e Uffici: 20032 CORMANO (Milano) Via Giotto, 19/21
Telefono +39-0266301938 r.a. - Telefax +39-0266301939
E-mail: info@germodis.com www.germodis.com

NEUTRALIZING LINE - TECHNICAL DATASHEET

DEACTIVATOR

DEACTIVATOR OF DISINFECTANT SOLUTIONS
CONTAINING GLUTARALDEHYDE
BEFORE DISPOSAL



INSTRUCTIONS FOR USE

PHASE 1:

Add by carefully mixing 50 ml (two 25 ml measuring caps) of DESACTIVATOR FASE 1, with a green label, for each liter of solution containing 2% of Glutaraldehyde (es. GERMOCID GLUTACARE).

An excessive dose does not cause damage.

Leave to react for 24 hours. At the end of this process the liquid will have assumed a different color.

PHASE 2:

Add 50 ml (two measuring caps) of DESACTIVATOR FASE 2, with a red label, to the liquid prepared during the first step.

Mixing thoroughly and allow to react for about an hour before disposal the product. The deactivated solution respects the environment.

PACKAGING

Internal Code:	Primary packaging:	Secondary packaging:
D200	One case containing: - 300 ml bottle of DESACTIVATOR FASE 1 with green label - 300 ml bottle of DESACTIVATOR FASE 2 with red label	Box containing 6 cases

All primary packaging is made of high polyethylene density (HDPE) according to the specifications provided by the Pharmacopoeia. This material does not contain latex and is perfectly compatible with all components of the formulation.



Germodis®
S.P.A.

Sede Legale Stabilimento e Uffici: 20032 CORMANO (Milano) Via Giotto, 19/21
Telefono +39-0266301938 r.a. - Telefax +39-0266301939
E-mail: info@germodis.com www.germodis.com

Capitale Sociale € 156.000,00 interamente versato - R.E.A. 497966 Cod. Fisc. e Part. IVA 00772350153 - Iscr. 125750 R.I. Canc. Trib. MI

NEUTRALIZING LINE - TECHNICAL DATASHEET

DEACTIVATOR

COMPOSITION

Phase 1: Contains Sodium Hydroxide.

Phase 2: Contains Hydrochloric Acid.

PRODUCT FORM

Colorless and odorless limpid liquid. Phase 1 with a basic pH. Phase 2 with an acid pH.

ACTIVITY

Deactivate Glutaraldehyde based disinfectants before disposal.

VALIDITY

5 years. The indicated period of validity refers to the product stored in its container and properly used and stored.

HOW TO STORAGE

Store in a cool, dry place, away from heat sources.

QUALITY CONTROL

The components (raw materials, containers, labels, etc.) and the processing steps of each lot are regularly and carefully checked internally following the procedures of the Organization Quality Management System certified UNI EN ISO 9001.

CONFIDENTIAL INFORMATION FOR PROFESSIONAL USERS