



**Germo**<sup>®</sup>  
S.P.A.

Capitale Sociale € 156.000,00 interamente versato - R.E.A. 497966 Cod. Fisc. e Part. IVA 00772350153 - Iscr. 125750 R.I. Canc. Trib. MI

Sede Legale Stabilimento e Uffici: 20032 CORMANO (Milano) Via Giotto, 19/21  
Telefono +39-0266301938 r.a. - Telefax +39-0266301939  
E-mail: info@germodis.com [www.germo.it](http://www.germo.it)

## LINEA GERMOCID - TECHNICAL DATASHEET

### GERMOCID TEC

GERMOCID TEC is a Medical Device CE 0546 class IIa in compliance with Directive 93/42/EEC.

GERMOCID TEC is an aqueous solution, ready-to-use, for quick cleaning and disinfection of all non-invasive medical devices surfaces. The practical dual function regulator allows the use with the classic spray or the foamer. The foam adheres to all surfaces avoiding excessive use of the product and limiting the formation of spray and the volatility of the components. Its formula optimizes the compatibility with equipment and all materials in use at nursing homes, dental clinics, medical surgeries, hospitals, beauty centers, etc. The particular formulation of GERMOCID TEC is suitable for alcohol-sensitive surfaces and materials such as: beds, cots, litters, facial masks, objects and plexiglass equipment. GERMOCID TEC is active against Gram- positive and Gram-negative bacteria, fungi, mycobacteria and viruses. It decontaminates, disinfects, degreases and cleans.



#### INSTRUCTIONS FOR USE

GERMOCID TEC is ready to use. Spray GERMOCID TEC directly on surfaces. Spread the product on the area of interest and leave it to act for 15' against bacteria and fungi, for 60' against mycobacteria and viruses; then wipe with a cloth. Rinse if necessary (eg. plexiglass).

#### PACKAGING

Internal Code:	Primary packaging:	Secondary packaging:
R620	750 ml bottle	Box containing 12 bottles

Barcode	ITF	Box size (mm)	Pallet composition
8009110019508	08009110019508	338 l x 253 p x 246 h	40 ctn (4 floors x 10)

All primary packaging is according to the specifications provided by the Pharmacopoeia. The materials do not contain allergens and are perfectly compatible with all components of the formulation.



**Germo**<sup>®</sup>  
S.P.A.

Capitale Sociale € 156.000,00 interamente versato - R.E.A. 497966 Cod. Fisc. e Part. IVA 00772350153 - Iscr. 125750 R.I. Canc. Trib. MI

Sede Legale Stabilimento e Uffici: 20032 CORMANO (Milano) Via Giotto, 19/21  
Telefono +39-0266301938 r.a. - Telefax +39-0266301939  
E-mail: [info@germodis.com](mailto:info@germodis.com) [www.germo.it](http://www.germo.it)

## LINEA GERMOCID - TECHNICAL DATASHEET

### GERMOCID TEC

#### COMPOSITION

100,0 g of product contain: Benzalkonium chloride 2,0 g; O-phenylphenol 0,2 g; coformulants; depurated water q.s. to 100,0 g.

#### PRODUCT FORM

Liquid with variable color (from colorless to pale green). Characteristic parfum.

#### ACTIVITY

The action of Benzalkonium chloride and O-phenilphenol guarantees a broad spectrum of action against Gram-positive and Gram-negative bacteria, fungi, mycobacteria and viruses (Adenovirus, HIV, HBV, HCV).

Norm	Activity	Contact Time
EN13727/EN14561/EN13697	Bactericidal	15 minutes
EN13624/EN14562/EN13697	Fungicidal	15 minutes
EN14348	Mycobactericidal	60 minutes
EN14476	Virucidal	60 minutes

#### WARNINGS

For professional use. Shake well before use. Do not use in presence of other soaps or detergents.

#### VALIDITY

3 years. The indicated period of validity refers to the product stored in its container and properly used and stored.

#### HOW TO STORAGE

Store in a cool, dry place, away from heat sources.

#### QUALITY CONTROL

The components (raw materials, containers, labels, etc.) and the processing steps of each lot are regularly and carefully checked internally following the procedures of the Organization Quality Management System certified UNI EN ISO 9001 and UNI EN ISO 13485.

#### AUTHORIZATIONS

Submitted request of authorization as Medical Device CE 0546 class IIa in compliance with Directive 93/42 / EEC as amended by Directive 2007/47 / EEC.

**CONFIDENTIAL INFORMATION FOR PROFESSIONAL USERS**