Sede Legale Stabilimento e Uffici: 20032 CORMANO (Milano) Via Giotto, 19/21

Telefono +39-0266301938 r.a. - Telefax +39-0266301939

E-mail: info@germodis.com www.germo.it

Capitale Sociale € 156.000,00 interamente versato - R.E.A. 497966 Cod. Fisc. e Part. IVA 00772350153 - Iscr. 125750 R.I. Canc. Trib. MI

LINEA GERMOCID - TECHNICAL DATASHEET

GERMOCID TEC WIPES

GERMOCID TEC WIPES is a Medical Device CE 0546 class IIa in compliance with Directive 93/42/EEC.

GERMOCID TEC WIPES are made of a non-woven fabric soaked with a SAQ-based aqueous solution for the practical and fast cleansing and disinfection of non-invasive medical devices. Their formula is compatible with all equipment and materials in use at hospitals, dental surgeries, medical clinics, etc.

The pleasantly scented wipes clean and disinfect reducing the risk of cross-contamination. The particular formulation of GERMOCID TEC WIPES is ideal for all alcohol-sensitive surfaces and materials such as: beds, stretchers, face masks, various objects and equipment also in plexiglass.





INSTRUCTIONS FOR USE

Remove a wipe from the packaging and rub on surfaces, objects and instruments to be cleaned and decontaminated ensuring that they are fully moistened. Leave it act for 15' against bacteria and fungi, for 60' against mycobacteria and viruses. It does not require rinsing.

PACKAGING

Internal Code:	Primary packaging:	Secondary packaging:
Q362	Jar containing 100 wipes	Box containing 6 jars

All primary packaging is made of high polyethylene density (HDPE) according to the specifications provided by the Pharmacopoeia. This material does not contain allergenics and is perfectly compatible with all components of the formulation.

Sede Legale Stabilimento e Uffici: 20032 CORMANO (Milano) Via Giotto, 19/21

Telefono +39-0266301938 r.a. - Telefax +39-0266301939

E-mail: info@germodis.com www.germo.it

Capitale Sociale € 156.000,00 interamente versato - R.E.A. 497966 Cod. Fisc. e Part. IVA 00772350153 - Iscr. 125750 R.I. Canc. Trib. MI

LINEA GERMOCID - TECHNICAL DATASHEET

GERMOCID TEC WIPES

COMPOSITION

100,0 g of the soaking solution contain:

Benzalkonium chloride 2,0 g; O-Phenylphenol 0,2 g; coformulants, depurated water q.s. to 100,0 g.

PRODUCT FORM

Perfumed soaked wipes.

ACTIVITY

The combined action of Benzalkonium chloride and O-phenylphenol guarantees a broad spectrum of action against Gram-positive and Gram-negative bacteria, fungi, mycobacteria and viruses (Adenovirus, HIV, HBV, HCV).

Norm	Activity	Contact Time
EN13727/EN14561/EN13697	Bactericidal	15 minutes
EN13624/EN14562/EN13697	Fungicidal	15 minutes
EN14348	Mycobactericidal	60 minutes
EN14476	Virucidal	60 minutes

WARNINGS

Keep out of reach of children. For professional use. Do not use in presence of other soaps or detergents.

VALIDITY

3 years. The indicated period of validity refers to the product stored in its container and properly used and stored.

HOW TO STORAGE

Store in a cool, dry place, away from heat sources.

QUALITY CONTROL

The components (raw materials, containers, labels, etc.) and the processing steps of each lot are regularly and carefully checked internally following the procedures of the Organization Quality Management System certified UNI EN ISO 9001 and UNI EN ISO 13485.

AUTHORIZATIONS

Submitted request of authorization as Medical Device CE 0546 class IIa in compliance with Directive 93/42 / EEC as amended by Directive 2007/47/EEC.

CONFIDENTIAL INFORMATION FOR PROFESSIONAL USERS