GERMOCID® ENZIMATICO

Detergente concentrato

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Concentrated detergent

for instruments
with proteolytic enzymes

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EN - Properties

GERMOCID ENZIMATICO is a proteolytic enzymatic detergent for washing and cleansing of the medical and dental instruments. It is efficacious also in presence of organic residues, even dry ones, and on persistent dirt: it can be used on glass, plastic and metal materials. For **PROFESSIONAL USE**, suitable for manual cleaning and also for use in ultrasound equipment. **Instructions for use:** Dilute **GERMOCID ENZIMATICO** at 0.6-10% in a liter of tap water, possibly warm, and mix thoroughly. Before dipping the instruments in the prepared solution, it is advised to decontaminate them with a specific product and rinse with cold water, then dip them in the solution of **GERMOCID ENZIMATICO** for 10 minutes; in presence of dry residues or persistent dirt extend immersion time to 1 hour, and if necessary use a brush.

Composition: Non-ionic surfactants 15-30%; anionic surfactants 5-15%; enzymes; perfume (Limonene, Citral), preservative (Benzisothiazolinone). **Warnings:** Causes serious eye damage. Causes skin irritation. Harmful to aquatic life with long lasting effects. Wash hands thoroughly after handling. Avoid release to the environment. Wear protective gloves / eye protection / face protection. **IF IN EYES:** Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Immediately call a **POISON CENTER / doctor**. Contains: Alkaline fatty alcohols. For professional use. Store in a cool and dry place, away from heat sources. After opening, if properly stored, the product keeps the validity indicated on the label.

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